

Measuring Instructions

CompreBoot™/FoamFoot™

1. Record **length** measurement **K** (length from 1st metatarsal head to heel), or desired length on the corresponding line. Select the boot size that matches according to row **K** on the **Sizing Chart** below.
2. Record **circumference** measurements **I, J,** and **L** on the corresponding lines.
3. Check to make certain that foot measurements **I, J,** and **L do not exceed** the adjustment range for the boot size selected in **STEP 1** (measurement **K** - **Sizing Chart** row **K**).
4. If any of the above foot measurements (**I, J,** and **L**) exceed the adjustment range of the boot size selected in **STEP 1 (K)** above, select the **next larger size** where all three measurements do not exceed the adjustment range of the boot.

CompreBoot™/FoamFoot™ - Sizing Chart (all measurements in cm)				
		Small	Medium/Large	X-Large/XX-Large
1. -	K	14-18	18-20	20-22
2. {	I	28-36	39 max	44 max
	J	22-26	30 max	33 max
	L	25-30	36 max	43 max

Measuring Instructions

MedaBoot™

**Digital photos are required when irregular shapes are present*

1. Record **length** measurement **K** (length from 1st metatarsal head to heel), or desired length on the corresponding line.
2. Record **length** measurement **M - Top of Foot**/3rd Metatarsal Head to Ankle Bend - on the corresponding line .
3. Record **circumference** measurements **I, J,** and **L** on the corresponding lines.