

Measuring Instructions

CompreFit® TH/FoamSleeve™ TH

The CompreFit® Universal, introduced in 2006, has experienced extremely positive reception from clinicians nationally. The unique properties of non-elastic compression allows the CompreFit® to provide a high working pressure (30-40 mmHg), while also providing a low resting pressure, making it ideal for day use and safe for night use. It is extremely durable, as well as easy to use and clean. Many physicians prefer CompreFit® for their patients.

1. Measure *length* from **5th Metatarsal** to **heel** on the outside of the foot.
Record in the corresponding box.
2. Measure *length* from the **lateral** malleolus (**outer ankle bone**) to **Bottom of Patella**.
Record in the corresponding box.
3. Measure *length* from the **lateral** aspect from **Top of Patella** to **Waist**.
Record in the corresponding box.
4. Measure *circumference* around the **5th Metatarsal**.
Record in the corresponding box.
5. Measure *circumference* around the **Ankle Bend and Heel**.
Record in the corresponding box.
6. Measure *circumferences* where indicated by the black dots from the **lateral** malleolus (**outer ankle bone**) to **Waist**.
Record in the corresponding lines (Left & Right).